

An instructional video is available at
www.doserite.info/video

Apply Medication into Anal Canal

1. Insert applicator into the anus up to the safety flange.
2. Press syringe plunger to apply medication.
3. Slightly rotate syringe to disperse medication before removing from anus.
4. Remove applicator from syringe and discard.

If using a reusable applicator see cleaning instructions.

5. Attach black syringe cap and store for future use.


Self-Medication Positions

Important: Insert the applicator until you reach the flange. This will ensure medication is delivered correctly.

With one leg elevated, insert applicator into the anus up to the flange.



Bend slightly forward and insert the applicator from behind. Ensure applicator is inserted up to the flange.

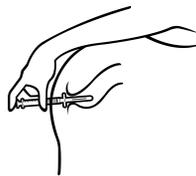

Laying on back, insert applicator into the anus up to the flange.



If you are unable to insert the applicator up to the flange, please contact your physician for alternative application options.

Intended Use

DoseRite is intended for use with topical medication prescribed by a physician to be applied to the anal canal. Consult your physician for medication and dosing instructions. DoseRite is intended for single person use. DoseRite product does not include medication. DoseRite is latex-free.

Contraindications

Use DoseRite only with medication prescribed by a physician. If a physician prescribes nitroglycerine medication (NTG), use gloves when handling these medications to avoid skin contact with NTG.

Warning: Notify Surgin of any serious incidents that occur while using this product.

Surgin Inc.
Scottsdale, AZ
Tel: 714.832.6300
Fax: 714.832.0300
www.doserite.info

DoseRite is a registered trademark of Surgin Inc.
DoseRite is patented and has patents pending.

For patent information: www.doserite.info/patents

Instructions for Use

Dosing and Application for the Anal Canal




www.doserite.info

What is DoseRite?

DoseRite is a new product to help patients accurately dose and apply medication within the anal canal.

Targeted Medication Delivery

DoseRite's patented applicator provides a simple and painless way to accurately apply your medication within the anal canal. The applicator's side slotted design ensures medication is applied directly to the anoderm for maximum absorption.

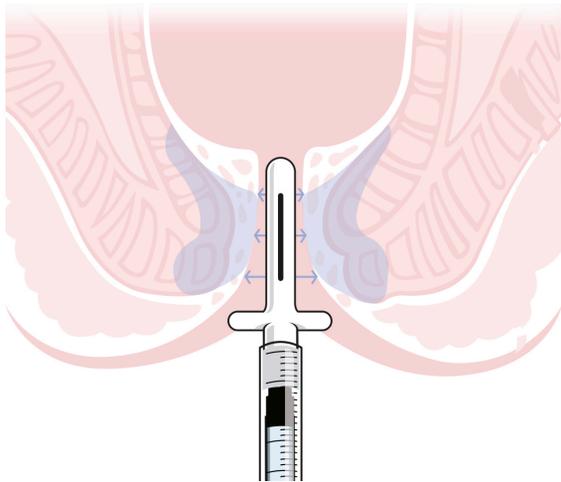

Accurate Dosing

DoseRite includes reusable dosing syringes to help create an accurate dose during each application. You have the option of using the dosing jar or purple tube adapter to transfer medication into the dosing syringe.

What's Included with DoseRite

Your DoseRite treatment kit includes a dosing jar, purple tube adapter, applicator tips and dosing syringes. If your medication was packaged in a tube or jar, see instructions for transferring your medication into the DoseRite jar.

Dosing Setup with a Jar

Remove the blue cap and press down on the clear bottom until it reaches the base of the jar.



If your medication is packaged in a jar use the wooden spatula to transfer medication into DoseRite jar.



Prime the Jar

1. Remove clear cap from medication jar adapter.

2. Prime the jar by pushing the bottom of jar upwards until medication appears inside the green adapter.



Create a Single Dose

1. Attach syringe to green medication jar adapter (A).

2. Fill the syringe with the correct dose of medication by slowly pulling up on the syringe plunger (B).

3. Remove syringe from jar and attach applicator (C).



Dosing Information

Check with your physician for the dose that is right for you. An instructional video is available at www.doserite.info/video


Dosing Setup with a Tube

1. Remove cap from medication tube and replace with purple adapter.




2. Remove clear cap from purple adapter.

Important – Prime the tube by squeezing until medication appears inside the purple adapter.

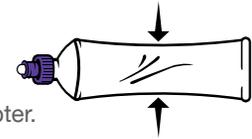

3. Remove black cap from syringe and attach syringe to purple adapter.

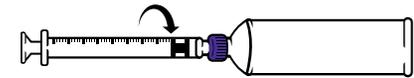

4. Fill the syringe with the correct dose of medication by slowly pulling on the syringe plunger.

5. Remove syringe from adapter and attach applicator (C).




Cleaning Instructions

1. After applying medication, remove the applicator from the syringe and clean with warm soapy water and rinse.



2. Fill the syringe with warm water and flush.

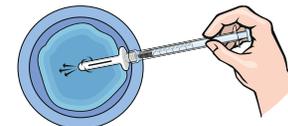

3. Refill the syringe with warm water and flush the applicator.



4. Before and after each use, insert the cleaning tool into the applicator and syringe to ensure the channels are clear.

